

Please read all instructions prior to use to prevent damage and/or injuries.



# Instruction for use

## Contact Info

info@optindustries.com

OPT Industries, Inc  
85 Hicks Avenue,  
Medford, MA 02155, USA

All designs, methods, and samples remain the property of OPT Industries, Inc ("OPT"). Recipient shall not disclose any confidential information to any third party without the written consent of OPT. OPT is providing these samples to the recipient for the sole purpose of evaluating the suitability of articles made from the designs for future purchase under a definitive supply agreement. Any other use of the samples or designs is strictly prohibited. Recipient shall destroy or return samples and copies of designs upon request of OPT. Recipient may give feedback to OPT that is incorporated into future designs, which feedback Recipient hereby assigns to OPT. OPT has not granted Recipient any licenses unless under a separate definitive agreement. The samples are being supplied to the recipient "as is" with no express or implied warranties.

## Disclaimer of Implied Warranties

Except as expressly provided in this agreement, all warranties, express or implied, including without limitation warranties of merchantability or fitness for a particular purpose or use, are hereby disclaimed.

## Limitation of Liability

OPT Industries will not be liable for any special, incidental or consequential loss or damages arising from use of any product.

## INTENDED USE

OPT's InstaSwab™ is sterile and intended for clinical sample collection. The applicator is suitable for short contact with the human body. Use as directed.

## STERILIZATION

The applicators have been sterilized with a 12 hour ETO process and individually packaged in airtight pouches.

## STORAGE

Store the applicator at room temperature in a dry location that is away from direct sunlight.

If the product is stored in different conditions, it will not maintain the stated specifications.

## DIRECTIONS FOR USE

Sample collection should be conducted by trained physicians or clinicians.

1. Instruct patient to exhale before inserting the applicator.
2. Instruct patient to breathe through the mouth during the swabbing.
3. Gently place the applicator bulb in the area of interest.
4. Hold the applicator in place for a few seconds for the liquids to absorb.
5. When encountering difficulty, do not force the insertion or rotation.

## WARNINGS

1. This product is manufactured from photo-curable acrylic resins. Be aware that it may have a slight, characteristic acrylic smell.
2. This product is for single use only; reuse may risk infection and/or inaccurate results.
3. This product is to be handled by trained personnel only.
4. Do not re-sterilize unused applicators.
5. Do not re-pack applicators.
6. Do not use if the applicator is visibly damaged.
7. Do not use excessive force, pressure or bending when collecting swab samples from patients as this may result in accidental breakage of the swab shaft.
8. Directions for use must be followed carefully. The manufacturer cannot be held responsible for any unauthorized or unqualified use of the product.
9. It must be assumed that all specimens contain infectious micro-organisms; therefore all specimens must be handled with appropriate precautions. After use, tubes and applicators must be disposed of according to laboratory regulations for infectious waste.